

SARS-CoV-2 Antigen Rapid Test Kit


(immuno-fluorescence)

With application in one for:

- Serum
- Nasopharyngeal extract (Swab)



the fast lane to the European IVD market

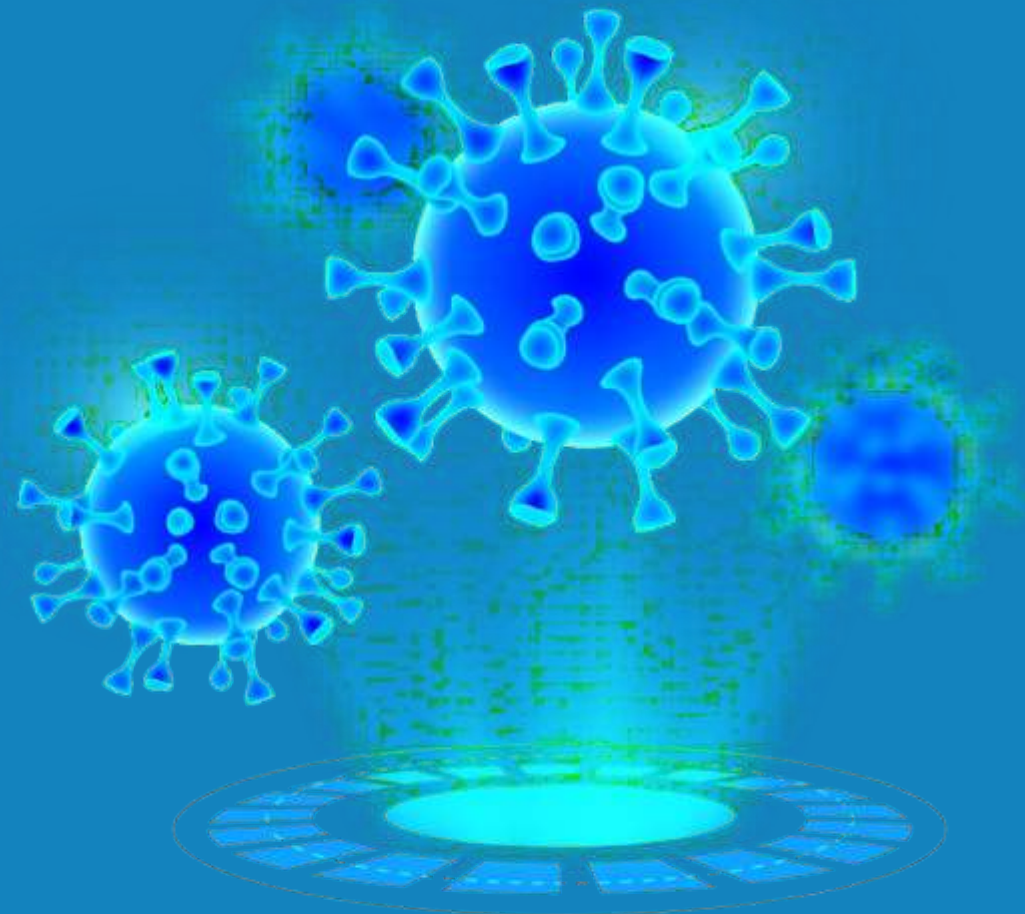



The SARS-CoV-2 Antigen Rapid Test Kit

With application for:

- Serum
- Nasopharyngeal extract (Swab)



The SARS-CoV- 2 Antigen Rapid Test Kit

The kit contains all material for the nasopharyngeal extract application

- Sterile swabs
- Extraction tubes
- Filter caps
- Buffer vials
- Test cassettes




The SARS-CoV-2 Antigen Rapid Test Kit

The kit contains all material for the nasopharyngeal extract application:

Kit content:

- 25 sterile swabs
- 25 extraction tubes
- 25 filter caps
- 2 buffer vials
- 25 test cassettes
- 1 Instruction for Use

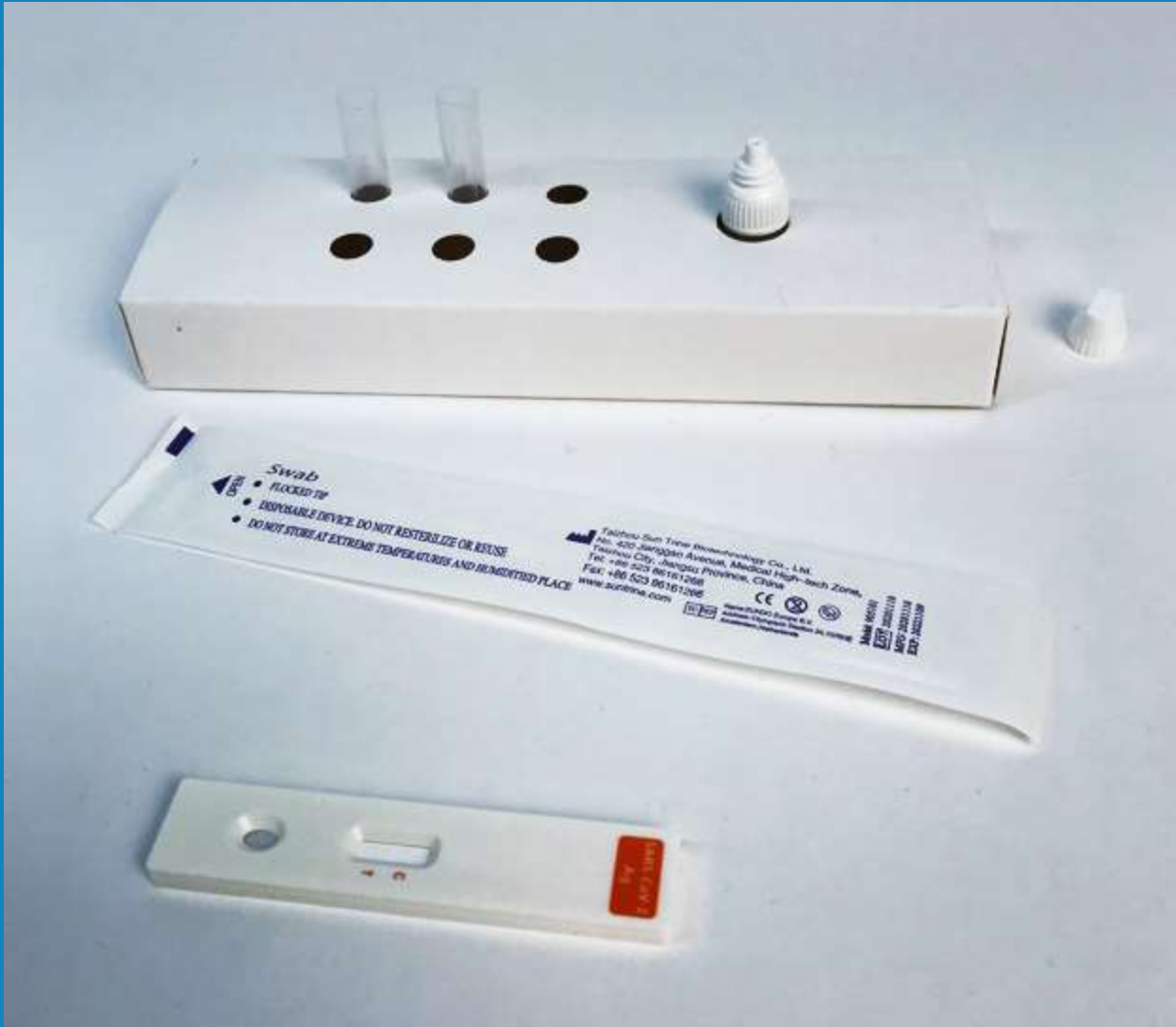

The SARS-CoV-2 Antigen Rapid Test Kit

The NP swab is ready to be used.

The cardboard box containing the swabs, can also be used as a rack.



The SARS-CoV-2 Antigen Rapid Test Kit

The swab has been used to collect the sample. The swab was swirled in the tube and the filter cap added.

The tube is then inverted and the operator adds three drops into the test cassette.



The SARS-CoV-2 Antigen Rapid Test Kit

After adding 3 drops onto the cassette the operator waits for 15 minutes.

First image shows the cassette after 1 minute showing the sample being drawn over the test strip.



After 15 minutes the test cassette is illuminated with the UV flash light.

The control band appears as a red-orange band in the inspection window of the test cassette.



The SARS-CoV-2 Antigen Rapid Test Kit

The UV flash light. (365nm)

The operator must add AA battery.

The flash light must be purchased separately. In some markets the flash light is provided free of charge with large ORDER.



the fast lane to the European IVD market

Info/Order Test Kits:



info@dialane.org
<https://dialane.org>



+41 81 250 7738

© 2020 Dialane AG.

Dialane AG, Löslweg 2, 7012 Felsberg, Switzerland+41 81 250 7738